### INSTRUCTIONS - PipeP Kit

Respiratory exercise system for inspiration and/or for expiration.  
 Equipped with one main connector, manometer connector and with 8 color coded resistors.  
 Resistor hole diameter: 1.5mm, 2.0mm, 2.5mm, 3.0mm, 3.5mm, 4.0mm, 4.5mm, 5.0mm.

CODE	DESCRIPTION
KM- 803	PipeP Kit

1. Remove the product from the package and check its components and integrity.
2. The main connector **A** is equipped with two valves, IN and OUT. The arrow indicates the direction of the airflow. For inspiration exercises, place a resistor in the IN side of the main connector as shown by **B**, while for expiration exercises, place a resistor in the OUT side of the connector. The diameter of the resistor is indicated on each color-coded resistor piece.
3. Use port **C** to attach the PIPEP to the face mask. The face mask is the interface between the patient and the respiratory exerciser.
4. Make sure that all connections are tight.

**WARNING:** This is a single-patient product. Do not swallow. **NON CONDUCTIVE PRODUCT. NON STERILE. LATEX FREE.**  
**CLEANING AND DISINFECTING:** Use a non aggressive detergent solution. Do not sterilize and do not use with more than one patient.  
 After the recommended period of use, discard the product according to normal procedures.  
**CAUTION:** The use of this device must be by or on order of a qualified physician. The patient must monitored at all times.

U.S. Federal law restricts this device to sale by or on the order of a physician

KOO (Shanghai) Industries Co., Ltd 100 Zhongde Road, Songjiang, Shanghai 201614, P.R.CHINA  
 Production site:KOO MEDICAL EQUIPMENT (Shanghai) Co., Ltd., 1 Songjiang Industrial Park, Songjiang Shanghai, 201613 P.R. CHINA

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Distributed in Asia Pacific by: Koo Asia Ltd, Unit 23, 9<sup>th</sup> Floor Goodwill Ind. Building No. 36-44, Pak Tin Pa Street, Tsuen Wan, N.T. Hong Kong  
 Made in CHINA

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Koo Europe S.r.l., Via della Praia 12/D, 10090 Buttigliera Alta (TO) - ITALY
KOO (Shanghai) Industries Co., Ltd 100 Zhongde Road, Songjiang, Shanghai 201614, KINA
Koo Europe Srl, Via della Praia 12 / D, 10090 Buttigliera Alta (TO) - ITALIEN